Radii Xpert LED attachments

INSTRUCTIONS FOR USE

Standard LED attachment Diagnostic LED attachment Orthodontic LED attachment Full Arch Bleaching LED attachment

Thank you for purchasing this LED attachment for the Radii Xpert LED curing light. For operation and safety for the actual Radii Xpert LED curing light, please refer to the manual contained in the kit. This instruction sheet only details safe operation in relation to the LED attachment purchased.

HOW TO CHANGE THE LED

To change the LED attachment, carefully pull off the LED attachment from the handpiece and click in a new LED attachment.

STANDARD LED ATTACHMENT

The Standard LED attachment provides light curing at contra angle. Status ring indicator: Blue.

Run time: 2, 10, 20 seconds as selected. The light can be terminated prior to this time by pressing the (top button) once. Beeps at completion.

Operation

- Remove the Radii Xpert handpiece from the charging dock and place a hygienic barrier sleeve.
- Note: Ensure the Radii Xpert barrier sleeve fits flush over the lens without wrinkling/folds. Seam of the sleeve should be positioned down the sides of the attachment and handpiece.
- Rotate the curing tip to desired position.
- Press and hold. A very low intensity light will illuminate to assist you to position the curing tip in the oral cavity. When the tip is in the correct position, release your finger tip and curing will commence. Status ring illuminates solid blue with the standard curing tip attached. The LCD displays 20, counting down to 0.

One long beep will sound when curing finishes and the status ring goes OFF.

- To start another 20 second cure, press (bottom button) and hold for 1-2 seconds. Numbers on LCD start flashing.
- Press (bottom button) to scroll through the options from last setting used. 2, 10 and 20 second selections are available.
- Press (bottom button) once to select. Numbers on LCD stop flashing.
- Discard barrier sleeve after each patient.

Clean and sanitise the handpiece. See 'CLEANING THE RADII XPERT' section in this manual.

How to set favourite curing time

- Press and hold for 1-2 seconds. Numbers on LCD start flashing.
- Press (bottom button) to scroll up through the options from last setting used.
- Press (bottom button) and hold for 1-2 seconds. Numbers on LCD stop flashing. Status ring illuminates solid green for 2 seconds with one short beep, to confirm selection is saved.
- A star appears on the bottom left of the LCD screen. Unit is ready to use.

How to delete a favourite setting and return to factory setting

Place the handpiece on the charging dock. Press (top & bottom button) simultaneously for 1-2 seconds, then status ring illuminates solid yellow for 4 seconds and returns to factory setting.

DIAGNOSTIC LED ATTACHMENT

The Radii Xpert Diagnostic LED attachment provides transillumination to assist clinical diagnosis. Transillumination is a very effective, inexpensive and quick way of detecting some dental problems, such as fractures, caries, interproximal caries, subgingival calculus, root canal openings, filling restoratives and non-vital teeth.

Status ring indicator: White.

Run time: 5 minutes. The light can be terminated prior to this time by pressing the (top button) once. Beeps at completion.

Operation

- Remove the Radii Xpert handpiece from the charging dock and click in the Diagnostic LED attachment. Place a hygienic barrier sleeve.
- Note: Ensure the Radii Xpert barrier sleeve fits flush over the lens without wrinkling/folds. Seam of the sleeve should be positioned down the sides of the attachment and handpiece.
- Rotate the tip to the desired position.

- Press (bottom button) for illumination to begin. The LCD displays 5 minutes, counting down to 0.
- One long beep will sound when illumination finishes and the status ring goes OFF.

The brightness of the light can be adjusted to the desired level. Low, medium or high intensity selections are available. The illumination of the Diagnostic LED attachment is a factory setting and cannot be changed.

- Press (bottom button) and hold for 1-2 seconds. (Intensity bar) on LCD will start flashing.
- Press (bottom button) to scroll through to your desired brightness.
- Press (bottom button) once to select. Numbers on LCD stop flashing.
- Press (bottom button) again to begin illumination.

When placed back in the charger, the brightness of the light will return to the factory default setting (medium).

- To set a favourite setting
- Press and hold for 1-2 seconds. (Intensity bar) on LCD starts flashing.
- Press (bottom button) to scroll through the options from last setting used.
- Press (bottom button) and hold for 1-2 seconds to save. (Intensity bar) on LCD stops flashing. Status ring illuminates solid green for 2 seconds with one short beep, to confirm selection is saved.
- A star appears on the bottom left of the LCD screen. Unit is ready to use.

How to delete a favourite setting and return to factory setting

Place the handpiece on the charging dock. Press (top & bottom button) simultaneously for 1-2 seconds, then status ring illuminates solid yellow for 4 seconds and returns to factory setting.

ORTHODONTIC LED ATTACHMENT

The Orthodontic LED attachment is designed for use by orthodontists to better suit their clinical needs in applications such as the efficient bonding of brackets.

Status ring indicator: Blue.

Run time: 5 minutes or 10 seconds as selected. The light can be terminated prior to this time by pressing the (top button) once. Beeps at completion. Factory default setting is 5 minutes.

Operation

- Remove the Radii Xpert handpiece from the charging dock and click in the Orthodontic LED attachment. Place a hygienic barrier sleeve.
- Note: Ensure the Radii Xpert barrier sleeve fits flush over the lens without wrinkling/folds. Seam of the sleeve should be positioned down the sides of the attachment and handpiece.
- Rotate the tip into desired position.
- Press (bottom button) to activate the curing cycle. LCD displays 5 minutes, counting down to 0.

One long beep will sound when curing finishes and the status ring goes OFF.

Changing the run time selection

- Press (bottom button) and hold for 1-2 seconds. Numbers on LCD will start flashing.
- Press (bottom button) to scroll through the run time selections of 10 seconds or 5 minutes. Press to select.

When placed back in the charger, run time selection will return to the factory setting (5 minutes).

How to set favourite curing time

- Press and hold for 1-2 seconds. Numbers on LCD start flashing.
- Press (bottom button) to scroll up through the options from last setting used.
- Press (bottom button) and hold for 1-2 seconds to save. Numbers on LCD stop flashing. Status ring illuminates solid green for 2 seconds with one short beep, to confirm selection is saved.
- A star appears on the bottom left of the LCD screen. Unit is ready to use.

How to delete favourite setting and return to factory setting

Place the handpiece on the charging dock. Press (top & bottom button) simultaneously for 1-2 seconds, then status ring illuminates solid yellow for 4 seconds and returns to factory setting.

FULL ARCH BLEACHING ATTACHMENT

The Full Arch Bleaching LED attachment is designed for acceleration of in-office tooth whitening materials such as Pola Office or Pola Office+.

Status ring indicator: Cyan.

Run time: 8, 10, 15 minutes as selected. The light can be terminated prior to this time by pressing the (top button) once. Beeps at half-way point for each selection. Factory default setting is 8 minutes.

Operating instructions

Prior to using the Radii Xpert with this LED attachment, please ensure the battery is fully charged.

- Insert LED attachment into the control section of the Radii Xpert.
- Cover unit with a barrier sleeve.
- Place light shield over the sleeved arch.

Once assembled, the unit can be operated by hand, or attached to the Radii Xpert Bleach Stand.

NOTE: Barrier sleeves are essential to prevent cross-contamination and provide protection for the light.

- Apply the in-office bleaching material as per manufacturer's instructions.
- Position the Radii Xpert bleaching light no more than 5-mm above the prepared teeth undergoing bleaching. If using the Radii Xpert Bleach Stand, use the various adjustable knobs to position accordingly.
- Press (bottom button) to activate the bleaching light. LCD displays 8 minutes, counting down to 0.
- One long beep will sound on completion and the status ring goes OFF. NOTE: The temperature emitted by the bleaching light reaches 37°C + 2°C (98.76°F + 3.5°F).
- Recharge the battery after completing one patient's treatment. NOTE: From a full charge, Radii Xpert gives six to seven 8-minute cycles before it requires battery charging.

Changing the run time selection

- Press (bottom button) and hold for 1-2 seconds. Numbers on the LCD will start flashing.
- Press (bottom button) again, to scroll through the run time selections of 8, 10 and 15 minutes. Press to select.

When placed back in the charger, run time selection will return to the factory default setting (8 minutes).

How to set a favourite setting

- Press and hold for 1-2 seconds. Numbers on LCD start flashing.
- Press (bottom button) to scroll through the options from last setting used.
- Press (bottom button) and hold for 1-2 seconds to save. Numbers on LCD stop flashing. Status ring illuminates solid green for 2 seconds with one short beep, to confirm selection is saved.
- A star appears on the bottom left of the LCD screen. Unit is ready to use.

How to delete a favourite setting and return to factory setting

Place the handpiece on the charging dock. Press (top & bottom button) simultaneously for 1-2 seconds, then status ring illuminates solid yellow for 4 seconds and returns to factory setting.

CLEANING THE RADII XPERT

The ideal way to protect patients from cross-infection is to utilise Radii Xpert barrier sleeves. Barrier sleeves are also essential to keep the Radii Xpert clean. When cleaning the Radii Xpert handpiece, ensure lens cap, battery pack, control section and any attachments are connected as one unit.

- LIQUID CONTACT WITH TERMINALS OF HANDPIECE AND CHARGER WILL CAUSE DAMAGE VOIDING WARRANTY.**
- LIQUID CONTACT WITH LED WILL CAUSE DAMAGE VOIDING WARRANTY. LIQUIDS MUST NOT ENTER ANY PART OF THE HANDPIECE OR CHARGER.**
- DO NOT SPRAY LIQUIDS DIRECTLY ONTO, OR APPLY COPIOUS AMOUNTS OF LIQUIDS TO THE RADII XPERT.**
- DO NOT AUTOCLAVE THE RADII XPERT.**

General surface cleaning

General surface cleaning can be conducted with anti-microbial surface disinfectant such as Glutaraldehyde, Chlorhexidine Gluconate, and 70% isopropyl alcohol. When applying the disinfectant to any component of the device, spray the disinfectant onto a piece of cloth and wipe over the external surfaces until completely clean.

SAFETY AND PRECAUTIONS

The safe operation of this unit depends on strictly following the operating instructions in this manual. The Manufacturer accepts no liability for any damage resulting from the use of this unit for any other purpose than the polymerization of dental composite material.

Automatic shut off function

Should the LED attachment become too hot due to excessive use, the status ring indicator will show one red flash, then automatically shut off and be unusable for approximately 1 minute until it cools down.

CAUTION!

Irradiation of the eyes bears an inherent health risk. Hence, the light must not be directed towards the eyes. Exposure must be restricted to the area of the oral cavity in which clinical treatment is intended. Suitable blue-light filtering safety goggles should be used. The Radii Xpert emits a high intensity light and intense light exposure of soft tissues (gingiva, oral mucosa and skin) should be avoided as this exposure may cause damage or irritation. The emitted light should be placed directly above the material to be applied or over soft tissue areas.

Do not use the Radii Xpert in patients or by users with:

- Heart pacemaker implants who have been advised to be cautious regarding their exposure to small electrical devices.
- A history of photo biological reactions (including individuals with cutaneous solaris or erythropoietic protoporphyria) or who are currently on photo-sensitising medication (including 8-methoxypsoralen or dimethylchlortetracycline).
- A history of catarrh of the eye. These people may be particularly sensitive to the exposure to light and should be discouraged from Radii Xpert treatment unless adequate safety measures, such as the use of protective goggles to remove blue light, are undertaken.
- A history of retinal disease. These people should seek advice from their ophthalmologist prior to operating the unit. In operating the Radii Xpert unit, this group of individuals must take extreme care and comply with all safety precautions (including the use of suitable light-filtering safety goggles).

WARRANTY PERIOD

SDI Ltd., the Manufacturer, extends the following warranty periods:

- Three (3) year warranty for the Radii Xpert Standard, Diagnostic and Orthodontic attachments.
- Two (2) year warranty for the Radii Xpert Full Arch Bleaching attachment.

Warranty covers defects in materials or workmanship to the original purchaser of this product. The Manufacturer agrees to correct any defects which develop within the warranty period, either by repair or replacement, at its option. This warranty is valid providing factory inspection indicates that any such defect developed during normal and proper use subject to the conditions below.

Warranty conditions – please read carefully

- Please fill out the warranty card included in the kit and send promptly back to SDI Ltd with proof of purchase. Failure to do so may void your warranty.
- Alternatively go to www.sdi.com.au/warranty to register your warranty.

- (Intense bar) on LCD will start flashing.
- Press (bottom button) and hold for 1-2 seconds. (Intensity bar) on LCD will start flashing.
- Press (bottom button) to scroll through to your desired brightness.
- Press (bottom button) once to select. Numbers on LCD stop flashing.
- Press (bottom button) again to begin illumination.

When placed back in the charger, the brightness of the light will return to the factory default setting (medium).

- To set a favourite setting
- Press and hold for 1-2 seconds. (Intensity bar) on LCD starts flashing.
- Press (bottom button) to scroll through the options from last setting used.
- Press (bottom button) and hold for 1-2 seconds to save. (Intensity bar) on LCD stops flashing. Status ring illuminates solid green for 2 seconds with one short beep, to confirm selection is saved.
- A star appears on the bottom left of the LCD screen. Unit is ready to use.

How to delete a favourite setting and return to factory setting

Place the handpiece on the charging dock. Press (top & bottom button) simultaneously for 1-2 seconds, then status ring illuminates solid yellow for 4 seconds and returns to factory setting.

ORTHODONTIC LED ATTACHMENT

The Orthodontic LED attachment is designed for use by orthodontists to better suit their clinical needs in applications such as the efficient bonding of brackets.

Status ring indicator: Blue.

Run time: 5 minutes or 10 seconds as selected. The light can be terminated prior to this time by pressing the (top button) once. Beeps at completion. Factory default setting is 5 minutes.

Operation

- Remove the Radii Xpert handpiece from the charging dock and click in the Orthodontic LED attachment. Place a hygienic barrier sleeve.
- Note: Ensure the Radii Xpert barrier sleeve fits flush over the lens without wrinkling/folds. Seam of the sleeve should be positioned down the sides of the attachment and handpiece.
- Rotate the tip into desired position.
- Press (bottom button) to activate the curing cycle. LCD displays 5 minutes, counting down to 0.

One long beep will sound when curing finishes and the status ring goes OFF.

Changing the run time selection

- Press (bottom button) and hold for 1-2 seconds. Numbers on LCD will start flashing.
- Press (bottom button) to scroll through the run time selections of 10 seconds or 5 minutes. Press to select.

When placed back in the charger, run time selection will return to the factory setting (5 minutes).

How to set favourite curing time

- Press and hold for 1-2 seconds. Numbers on LCD start flashing.
- Press (bottom button) to scroll up through the options from last setting used.
- Press (bottom button) and hold for 1-2 seconds to save. Numbers on LCD stop flashing. Status ring illuminates solid green for 2 seconds with one short beep, to confirm selection is saved.
- A star appears on the bottom left of the LCD screen. Unit is ready to use.

How to delete favourite setting and return to factory setting

Place the handpiece on the charging dock. Press (top & bottom button) simultaneously for 1-2 seconds, then status ring illuminates solid yellow for 4 seconds and returns to factory setting.

Full Arch Bleaching LED Attachment

The Full Arch Bleaching LED attachment is designed for acceleration of in-office tooth whitening materials such as Pola Office or Pola Office+.

Status ring indicator: Cyan.

Run time: 8, 10, 15 minutes as selected. The light can be terminated prior to this time by pressing the (top button) once. Beeps at half-way point for each selection. Factory default setting is 8 minutes.

Operating instructions

Prior to using the Radii Xpert with this LED attachment, please ensure the battery is fully charged.

- Insert LED attachment into the control section of the Radii Xpert.
- Cover unit with a barrier sleeve.
- Place light shield over the sleeved arch.

Once assembled, the unit can be operated by hand, or attached to the Radii Xpert Bleach Stand.

NOTE: Barrier sleeves are essential to prevent cross-contamination and provide protection for the light.

<p>Made in Australia by SDI Limited Bayswater, Victoria 3153 Australia 1 800 337 003 Austria 0800 0225 5734 Brazil 0800 770 1735 France 0800 0225 5734 Germany 0800 1005759 Ireland 0800 0225 5734 Italy 0800 0225 5734 New Zealand 0800 734 034 Spain 00800 0225 5734 United Kingdom 0800 0225 5734 USA & Canada 1 800 228 5166 www.sdi.com.au</p>	<p>ENGLISH</p>
---	----------------

<p>SDI Germany GmbH Hansestrasse 85 51149 Cologne Germany +49 2203 9255 0 Last Revised: 10-2017</p>	<p>DEUTSCH</p>
---	----------------

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